

Driver Installation

Follow the steps below to uninstall previous driver versions and to install the new driver included in this package.

- 1) Unplug in USB cable with TD4237 device.
- 2) Remove old driver that have previously been installed. For Windows systems, open the Add or Remove Programs window from the Control Panel. Next, select each “CP210x USB to UART Bridge Controller” and “TaiDoc BGM Device (Driver Removal)” entry and click on Remove.
- 3) Run the driver executable, TaiDocVCPInstaller.exe, to extract all of the device drivers included with this release (Windows, Macintosh and Linux), and the computer must be restart.
- 4) Connect Computer with TD4237 Device using USB Cable, and computer screen will show “Found New Hardware Wizard”. Use the wizard to install the drivers by selecting “Install the software automatically (Recommended)” directly. Next Operator system will show warning window and please select “Continue Anyway” button to continue.
- 5) Check the driver install success. For Windows systems, make a sure “TaiDoc BGM Device” item exists in Port of the Device Manager.

